

# CARUMED

*Because we care!*

## Digital Fever thermometer SC 44 Flex



SC 44 Flex is a maximum thermometer, meaning after 30 seconds measuring it will show the highest measured value. The end of the measuring time is marked by a beep.

The latest value is being stored. To see the last stored value, press the on button and hold it for three seconds. When the button is released again, the thermometer is ready to measure a new value. The thermometer is delivered in a transparent plastic box for storage.

Available in three varieties: duck, frog and penguin.

- Flexible tip
- 30 sec measure time
- Fever alarm
- Saves last value
- Signal tone
- Water tight

For an accurate measuring, it's recommended to measure rectally. This method is specifically recommended for infants and children. Insert the tip of the thermometer up to 2 cm in the rectum. When the signal tone beeps, the measuring is done.

Alternative method of measuring is in the mouth cavity (under the tongue). Make sure the mouth is closed during the measuring time, and that the tip of the thermometer is in contact with the mouth tissue the whole time. Otherwise the measure result won't be correct.

Axillary measuring (in the arm pit) is not recommended. The risk of getting incorrect values is too high.

Range	32,0°C - 43,9°C
Resolution	0,1°C
Precision	±0,2°C < 35,5°C ±0,1°C at 35,5°C-42°C ±0,2°C > 42°C
Display	Three digit LCD
Storage conditions	0-50°C, humidity 15-95%
Battery	1 * LR41 (1,5V)
Battery indicator	Change battery when the "empty battery" symbol is shown
Shut off	Auto shut off after 3 min
Dimensions	154*21*12mm
Weight	18g with battery

### Article Numbers

240310	Unassorted varieties
240311	Duck
240312	Frog
240313	Penguin

**CARUMED**

Företagshusvägen 5, 244 93 Kävlinge, Sweden

Tel: +46 (0) 46 - 73 41 00

info@carumed.se, www.carumed.se